Cube-Fix

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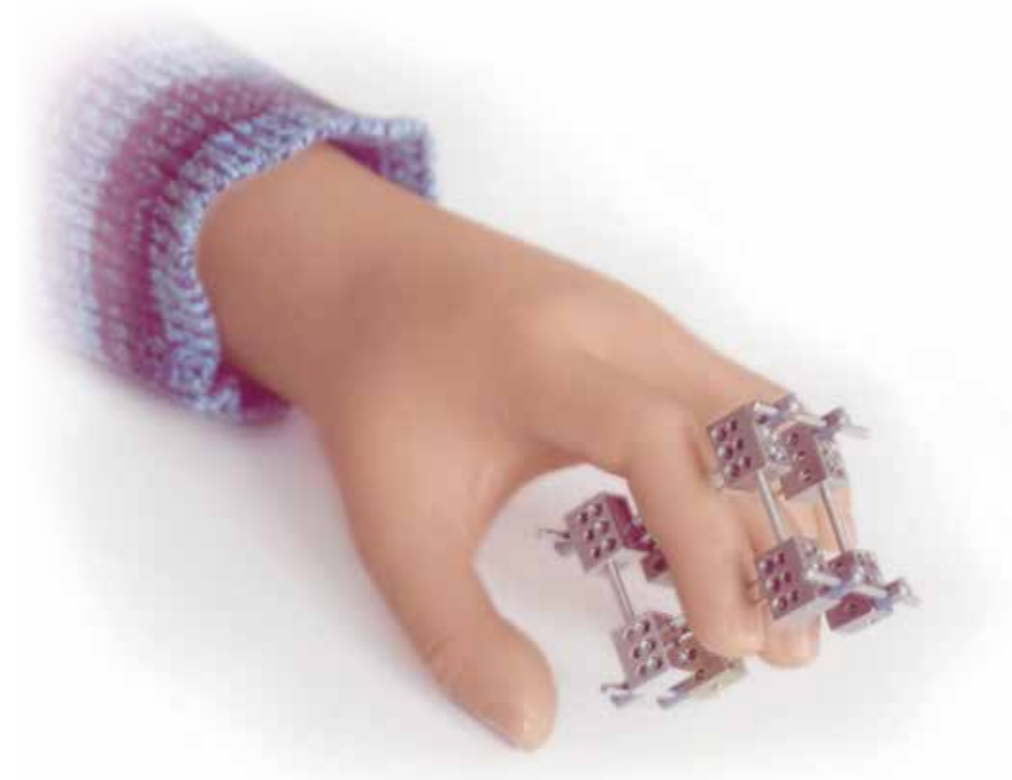
## modular fixation device **Cube-Fix**

for soft tissue and bone distraction treatment  
in infants and toddlers

The cube fix system is a light modular mini fixing device with a removable distraction unit. It can be adapted to the small anatomical structures of children's hands, is applicable in malformation and trauma cases

### Indications

- soft tissue distraction in complex syndactylies
- distraction lengthening of metacarpals and phalanges
- used as external fixator for stabilization of fractures and bone defects

### Contraindications

- skin infection

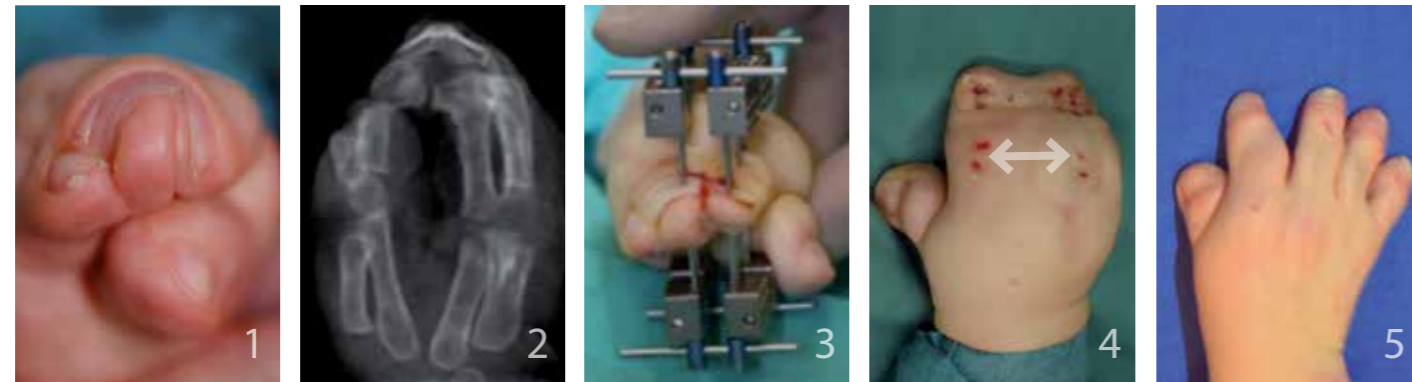
### Advantages

- loading with small K-wires from 0.8 mm on and with threaded pins is possible
- adaptable to smallest infant structures while preserving growth plates
- modular construction makes variable use possible
- low weight
- removable distraction unit
- experience of more than 25 years

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
Before treatment you find an osseous fusion of the distal phalanges (pictures 1/2). After separation of adjacent fingers little amount of own skin remains. Tissue for pulp plasties is lacking. Picture 4 shows the gain of skin and nailbed width after distraction, creating well shaped distal phalanges and pulps. Picture 5 four years after surgery.

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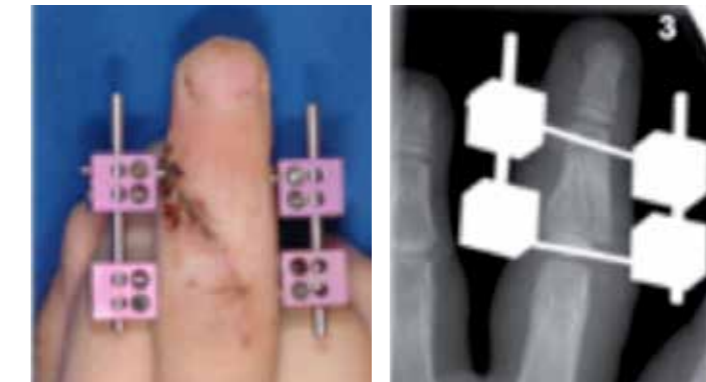
The cubes offer to be equipped with small K-wires. That is helpful for bone fractures of small children. The unstable open comminuted fracture of the middle phalanx in a 7 year-old boy is positioned in correct axial alignment and stabilized by the Cube-Fix.

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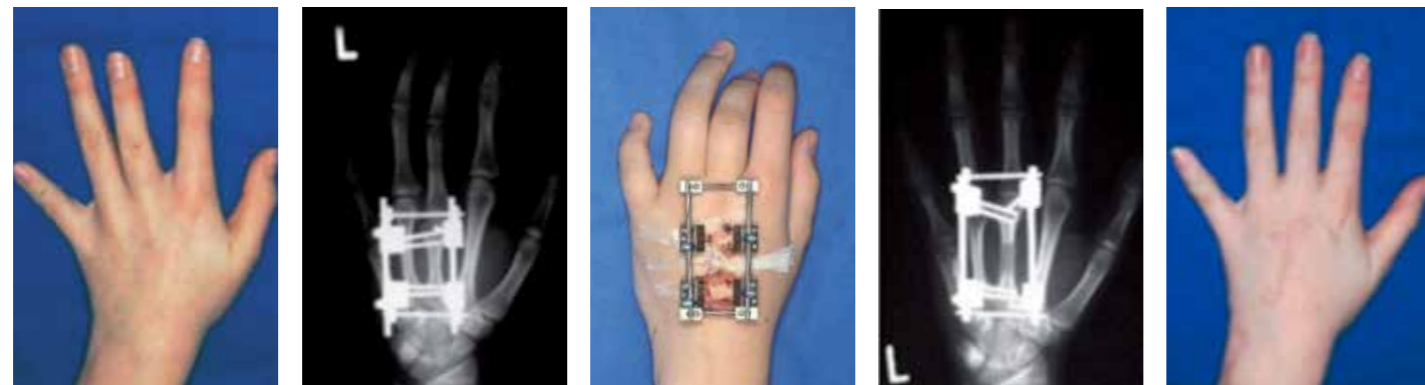Lengthening by distraction of the 3rd metacarpal in brachymetacarpia

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
The frame-styled Cube-Fix is suitable also for lengthening of metacarpals, here of the 3rd metacarpal bone.

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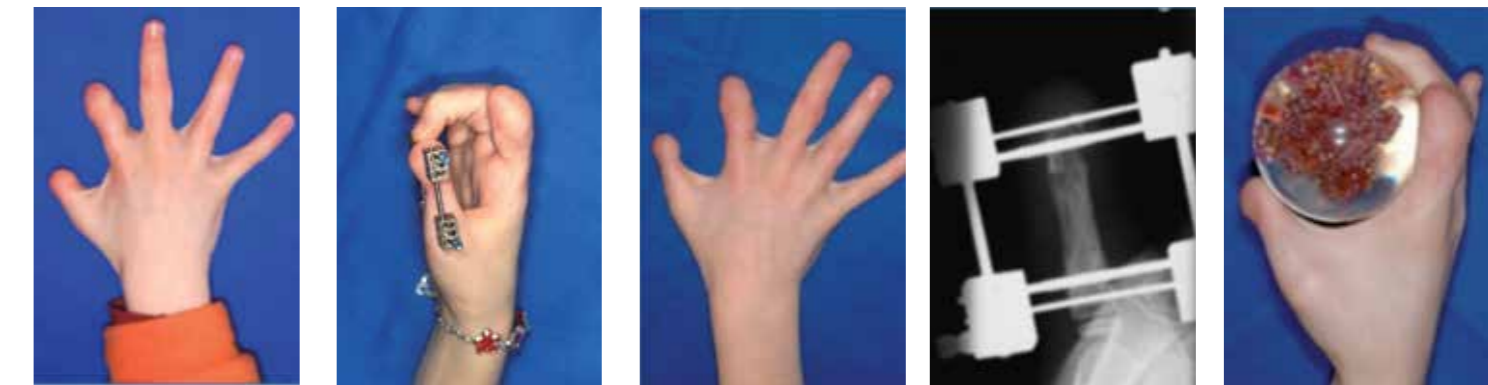Lengthening of the partially amputated thumb in constriction band syndrome

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A lengthening of 14 mm of the proximal phalanx was achieved.

The longer thumb improves grasping of larger objects and pincer grip between thumb and fingers.

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Mounting for soft tissue distraction in complex syndactyly  
Nailband and synostosis of the distal phalanges of long and ring finger are cut

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The first pin with middle thread is dorsally inserted into the center of the proximal phalanx perforating the palmar skin.

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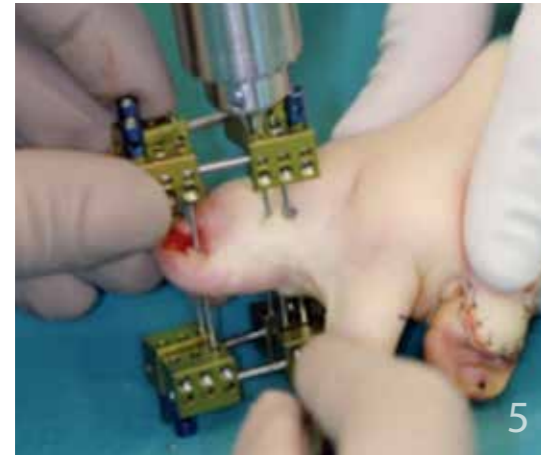
A second pin with middle thread is inserted into the adjacent finger at the same level. Then K-wires of 1 mm are inserted distally into the metaphysis of the distal phalanges.

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
Two 3 hole-pin cubes connected by a connecting rod are put onto these pins and fixed at the side with set screws.

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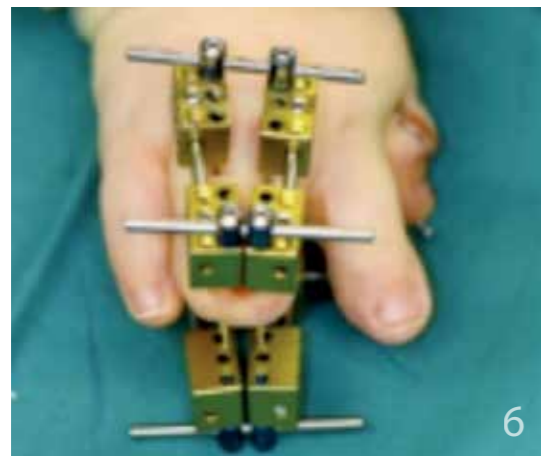Every finger is equipped with a cube and a connecting rod at the dorsal and palmar side.  
Each cube is secured against rotation by a second K-wire  
(K-wire of 1 mm at the proximal, 0.8 mm at the distal phalanx).

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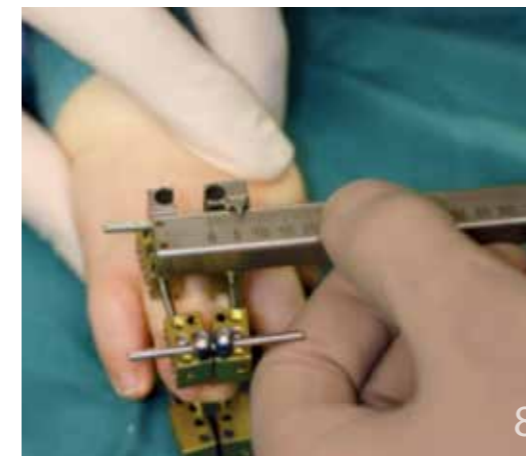
Connecting rods are inserted into the applied blue connection pins transversally at the dorsal and palmar side.  
Finished installation from the dorsal aspect.

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Finished installation  
from the distal and palmar aspect.

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
For soft tissue distraction the distractor is put on, a set screw of the blue connection pin is released,  
and distraction performed by one full turn (i.e.a gain in width of 0.5 mm). The set screw is tightened again and  
the distractor removed. Distraction has to be done at all 4 connecting rods.

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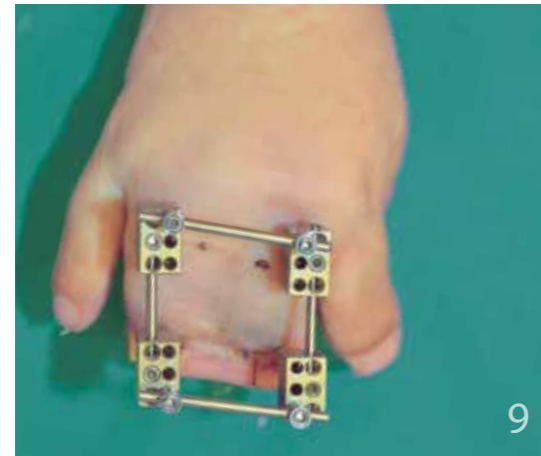After four weeks of distraction the goal of reaching 20 mm width was achieved

Elternset für  
modularen Klötzchenfixateur **Cube-Fix**  
parents' set  
for the modular fixation device **Cube-Fix**

Elternset für Cube-Fix mit folgendem Inhalt: 11-79000  
1 Distraktor Art.-Nr. 11-70000  
1 Schraubendreher Art.-Nr. 11.58.007  
10 Madenschrauben Art.-Nr. M3T  
parents' set for Cube-Fix with following content: 11-79000  
1 distractor ref.-no. 11-70000  
1 screwdriver ref.-no. 11.58.007  
10 set screws ref.-no. M3T

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**FIELD OF USE**

This brochure contains information about external fixation systems for the hand, especially for the treatment of fractures with joint involvement of the phalanges. Also for the soft tissue and bone distraction in complex syndactylies and fractures of the metacarpal bones and phalanges.

**GENERAL WARNINGS AND PRECAUTIONS**

As for now, none. Although it has to be pointed out that, a multiple use of external fixation systems may lead to a weakening of the connections within the system. Spherical, clamped connections are pointed out especially. Furthermore problems according to the perforation area have to be pointed out. Bacteria may enter the body through this perforation, which can lead to infections. Infections may be triggered also by insufficient or false cleaning of the perforation area. Furthermore, it is known that pins, nails and screws made of titanium show a lower infection rate than their steel pendant.

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Schraubendreher Klinge Tx6 TX6AO  
screw driver blade Tx6 TX6AO

Schraubendreher Handgriff SHGM34  
screw driver handle SHGM34

Innovations Medical Badstraße 11, 78532 Tuttlingen

Innovations Medical GmbH  
Badstraße 11, 78532 Tuttlingen, Germany  
Phone: +49 7461 / 96642 - 0  
Fax: +49 7461 / 96642 - 82  
Mail: [info@innovations-medical.de](mailto:info@innovations-medical.de)  
Web: [www.innovations-medical.de](http://www.innovations-medical.de)