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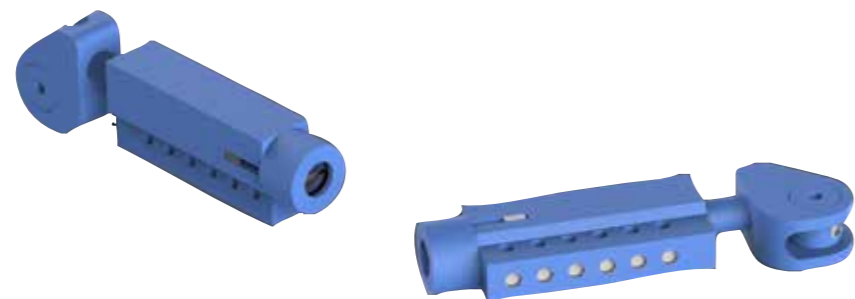
Indications

- intraarticular fractures of the middle phalanx base with and without luxation

Contraindications

- irreducible luxation fracture
- infected, lacerated skin areas

Advantages

- variable distraction force
- easy K-wire application
- short surgery time
- reimbursement by insurance as single-use product

Clinical consulting, design assistance

- Dr. H.-W. Bouman, ATOS Klinik, Heidelberg
- Prof. Dr. F. Capanni, Univ. of Appl. Sciences, Ulm

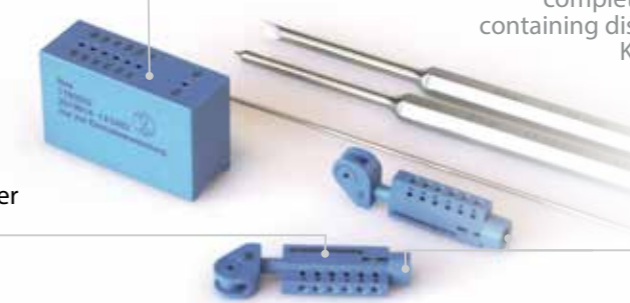
dynamic finger joint distractor
for the treatment of intraarticular fractures
of the PIP-joint

Bohrere zur sicheren
K-Draht-Platzierung
drill gauge for accurate
K-wire placement

Komplett-Set für Einmalanwendung:
Distraktoren, Bohrschablone,
K-Drähte und Schraubendreher
complete single use surgery set
containing distractors, drilling gauge,
K-wires and screwdrivers

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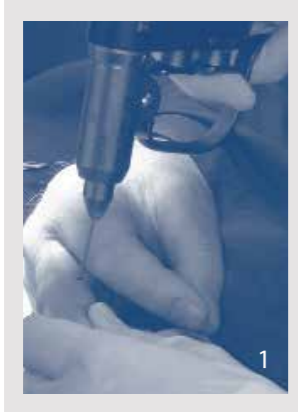
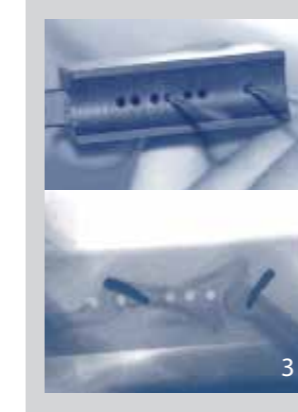
Drill first K-wire into the “rotation center” of the head of the proximal phalanx under image intensifier control.

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Front and side x-ray control of correct positioning of the wire.

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Using the drilling gauge insert second K-wire parallel and distal to the first K-wire. (The numbering on the distractor and drilling gauge must coincide!)
To avoid misrotation of the PIP joint, it should be bent 90°.

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
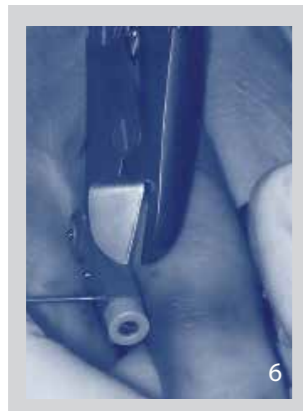
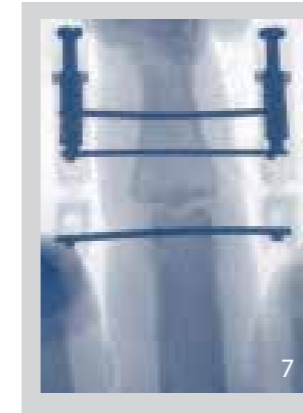
safety instruction

Tighten set screws for K-wires carefully. Over-tightening of the screws damages the thread in the distractor, a fixation of the wire is not longer possible.
Single use only. See reprocessing instructions.

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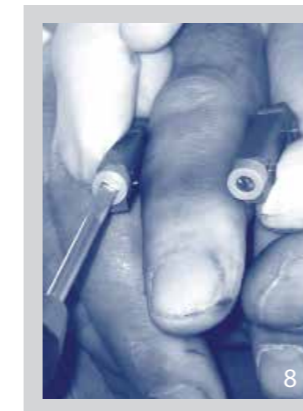If necessary, increase the spring tension of both distractors by turning the screws counterclockwise, control of finger extension and flexion

Innovations Medical Badstraße 11, 78532 Tuttlingen

Innovations Medical GmbH
Badstraße 11, 78532 Tuttlingen, Germany
Phone: +49 7461 / 96642 - 0
Fax: +49 7461 / 96642 - 82
Mail: info@innovations-medical.de
Web: www.innovations-medical.de